

Instructions For Use Eco+ Visor Mask



IMPORTANT

Read this 'Instructions for Use' document before handling. This product is for professional use only.

IDENTIFICATION

Product Name	Eco+ Visor Mask
Company Name	Perfection Plus Ltd
Company Address	 6, Westwood Court, Totton, Hants. SO40 3WX UK
Company Phone No.	+44 (0) 2380 866 677

PRODUCT CODE

REF	PP095/1050G	Eco+ Visor Mask Set Green Frame & 10 Shields
REF	PP095/1050TB	Eco+ Visor Mask Set Blue Frame & 10 Shields
REF	PP095/1050TP	Eco+ Visor Mask Set Pink Frame & 10 Shields
REF	PP095/1050W	Eco+ Visor Mask Set White Frame & 10 Shields
REF	PP095/1050Y	Eco+ Visor Mask Set Yellow Frame & 10 Shields
REF	PP095/1051	Eco+ Visor Replacement Shields pk 25
REF	PP095/1052	Eco+ Visor Replacement Shields pk 12

INSTRUCTIONS FOR USE

ASSEMBLY

Peel off the protective coating from both sides of the disposable plastic visor. Firmly attach the plastic visor shield using the plastic clips on the arms of the frame.

CLEANING

- The frame can be washed with standard soap and rinsed thoroughly. To disinfect use a validated steam autoclave for 20 minutes at 121°C.
- The visor shield can be cleaned and reused. Wash using standard soap and rinse thoroughly using standard soap. The use of an anti-fogging spray is recommended. The visor shield can be disinfected in a thermal disinfectant – Do not exceed 93°C.
- After cleaning check the visor shield for any distortion or damage which may affect transparency. Discard if vision will be adversely affected.

WARNING!

THIS PROTECTIVE VISOR MASK IS ONLY TO BE USED AS A BARRIER FOR PROTECTION AGAINST LIQUID SPLASHES

DO NOT use alcohol-based products to clean the visor frame or visor shields.



www.perfectionplus.com

Issue Date: April 2020