

Instructions for Use Gates Glidden Drills



IMPORTANT

Read this 'Instructions for Use' document before handling. This product is for professional use only.

Gates Glidden Drills are made from high grade stainless steel and are intended to be used to enlarge the coronal access. They are provided in a non-sterile state.

IDENTIFICATION

Product Name	Gates Glidden Drills, pack of 6
Company Name	Perfection Plus Ltd
Company Address	6 Westwood Court, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

PRODUCT CODE

REF	PP004/4001 Gates Glidden Drills RA 1 - 0.50mm Pk 6
REF	PP004/4002 Gates Glidden Drills RA 2 - 0.70mm Pk 6
REF	PP004/4003 Gates Glidden Drills RA 3 - 0.90mm Pk 6
REF	PP004/4004 Gates Glidden Drills RA 4 - 1.10mm Pk 6
REF	PP004/4005 Gates Glidden Drills RA 5 - 1.30mm Pk 6
REF	PP004/4006 Gates Glidden Drills RA 6 - 1.50mm Pk 6
REF	PP004/4007 Gates Glidden Drills RA 1-6 - Assorted Pk 6

INDICATIONS FOR USE

For use by dental professionals during a root canal treatment to enlarge the coronal access.

CONTRA INDICATIONS

None known.

INSTRUCTIONS FOR USE

1. Before cleaning the instrument, ensure there is no sign of damage, deformation or rust. Dispose of any instrument which is damaged.
1. It is essential that the instruments are cleaned and sterilised before first use and before reuse. Brush the instrument with a nylon brush. Ultrasonic sounds with a decontamination solution, rinse thoroughly and dry. Sterilise in a sterilisation pouch, for 18 minutes in a steam autoclave at 134°C.
2. Once sterilised, store in a sterilisation pouch such as Perfection Plus Self Seal pouch 002/5000.
3. Select the appropriate size for the treatment.
4. Connect to a contra angle hand piece. Select appropriate engine speed and torque. Test rotation before use to ensure the drill is securely fixed.
5. Irrigate and introduce the instrument into the canal, applying reciprocating and vertical filling movements.

WARNING!

It is recommended that PPE is worn when using this product, eye protection and gloves.
This product is sharp, avoid puncturing finger.
Dispose of product if it is damaged or has reduced efficacy.



STORAGE

Store in clean, dry conditions, protected from damp and corrosive products.



Batch code/Expiry Date

The life span is estimated at 3 years from the date of manufacturing but can vary depending on the use (cutting, conditions) on the frequency of use, and on the number of sterilisation cycles.

Symbols Used

	Reusable Medical Grade Stainless Steel
	Autoclave at 134°C

SPECIALLY FORMULATED FOR USE IN DENTISTRY



www.perfectionplus.com

CE
0123