

Instructions for Use Prophylaxis Paste



IMPORTANT

Read this 'Instructions for Use' document before handling. This product is for professional use only.

For treatment of dentinal and post-operative sensitivity.

IDENTIFICATION

Product Name	Prophylaxis Paste
Company Name	Perfection Plus Ltd
Company Address	6 Westwood Court, Brunel Road, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

PRODUCT CODE

REF PP010/0001 – Medium Mint	REF PP010/0046 – Coarse Mint
REF PP010/0002 – Medium Orange	REF PP010/0047 – Coarse Orange

INDICATIONS FOR USE

The prophy paste is thixotropic: so if desired, the fluidity of the paste can be increased by gentle spatulation before use. Replace lid after use. For best results use with prophy type handpiece and cup/brush combination.

INSTRUCTIONS FOR USE

Fill cup/brush with paste and run at low speed over each tooth surface, making sure that the edge of the cup/brush goes below the free margin of the gingival sulcus. Repeat this procedure for each tooth until all exposed surfaces have been treated. Un-waxed Dental Floss may be used to remove any excess paste through interproximal areas.

STORAGE

Replace lid after use.
Store in cool, dry conditions.



BATCH CODE/EXPIRY DATE

The batch number should be quoted in all correspondence. The batch code gives an open date of manufacture month, year and day format, with a numerical suffix to uniquely identify the batch of material. Do not use after expiry date.

SPECIALLY FORMULATED FOR USE IN DENTISTRY



www.perfectionplus.com

EC REP Advena Ltd
Tower Business Centre
2nd Floor Tower Street
Swatar, Malta

