

Instructions For Use Protect+ Visor Mask Set



IMPORTANT

Read this 'Instructions for Use' document before handling. This product is for professional use only.

IDENTIFICATION

Product Name	Protect+ Visor Mask Set
Company Name	Perfection Plus Ltd
Company Address	 6, Westwood Court, Totton, Hants. SO40 3WX UK
Company Phone No.	+44 (0) 2380 866 677

PRODUCT CODE

REF	PP011/0050	1 Frame and 6 shields
REF	PP011/0051	12 Replacement shields

INSTRUCTIONS FOR USE

ASSEMBLY

Peel off the protective coating from both sides of the disposable plastic visor. Firmly attach the plastic visor shield to either side of the frame using the crocodile clips on the arms of the frame.

CLEANING

- The frame can be washed with standard soap and rinsed thoroughly. To disinfect use a validated steam autoclave for 20 minutes at 121°C.
- The visor shield can be cleaned and reused. Wash using standard soap and rinse thoroughly using standard soap. The use of an anti-fogging spray is recommended. The visor shield can be disinfected in a thermal disinfectant – Do not exceed 93°C.
- After cleaning check the visor shield for any distortion or damage which may affect transparency. Discard if vision will be adversely affected.

WARNING!

THIS PROTECTIVE VISOR MASK IS ONLY TO BE USED AS A BARRIER FOR PROTECTION AGAINST LIQUID SPLASHES

DO NOT use alcohol-based products to clean the visor frame or visor shields.



www.perfectionplus.com

Issue Date: April 2020