

Instructions for Use ConnectED**IMPORTANT**

Read this 'Instructions for Use' document before handling. This product is for use by dental professionals only.

IDENTIFICATION

A HEMA-free all in one adhesive to etch, prime and bond in one step. It offers excellent bond strength to enamel and dentine. Bond durability and biocompatibility are outstanding.

Product Name	ConnectED
Company Name	Perfection Plus Ltd
Company Address	 6 Westwood Court, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

PRODUCT CODE

REF PP010/0120 - ConnectED Kit 6ml bottle, 50 Appli+ tips and 20 mixing wells
REF PP010/0121 - ConnectED 6ml bottle – Triple bottle pack
REF PP010/0122 - ConnectED 6 bottle

INDICATIONS FOR USE

Suitable for use with composites and compomers.

Side Effects:

Isolated cases of contact allergies with products of similar composition have been reported. To avoid possible pulpal reactions, protect the exposed dentine in the pulpal area with an appropriate lining material (preferably with a calcium hydroxide cavity liner such as PerfectaLINE).

Interactions:

Avoid cavity liners which might inhibit polymerisation due to their components, e.g. phenolic substances such as zinc oxide eugenol.

INSTRUCTIONS FOR USE**CONTRAINDICATIONS:**

Exposed pulp, pulpitis, and known allergy to methacrylate resin.

1. Clean the tooth surface and prepare the cavity following conventional methods. Isolate with rubber dam if appropriate.
2. Apply calcium hydroxide cavity liner (PerfectaLINE) in deep cavities where the remaining dentine is thin.
3. Apply a thin layer of adhesive and leave undisturbed for 10 seconds. Dry gently with oil-free air for 5 seconds.
4. Light cure with Perfect+ LED05 curing light for 5 seconds. Cavity is now ready for filling with composite or compomer restorative.

WARNING!

Close bottle after each use. Keep material out of the reach of children. In case of direct contact with oral mucosa, rinse with plenty of water. Harmful if swallowed. Irritating to eyes and skin. May cause sensitisation by skin contact. In case of contact with eyes, rinse immediately with water and seek medical advice. After contact with skin, wash immediately with plenty of soap and water. Wear suitable protective clothing, gloves and eye/face protection. Highly flammable liquid.

**STORAGE**

Store in a refrigerator (1-10°C).

Caution: Do not expose to direct sunlight or other heat sources. Do not freeze.

**Batch code/Expiry Date**

The batch number should be quoted in all correspondence. The batch code gives an open date of manufacture month, year and day format, with a numerical suffix to uniquely identify the batch of material. Do not use after expiry date shown.

SPECIALLY FORMULATED FOR USE IN DENTISTRY

