

Instructions for Use Paste Fillers



IMPORTANT

Read this 'Instructions for Use' document before handling. This product is for professional use only.

Paste Fillers (Lentulo) are made from high grade stainless steel, the shank is an alloy of copper, nickel and zinc. and are intended to be used to enlarge the coronal access. They are provided in a non-sterile state.

IDENTIFICATION

Product Name	Paste Fillers (Lentulo) Pack of 6
Company Name	Perfection Plus Ltd
Company Address	6 Westwood Court, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

PRODUCT CODE

REF PP004/1300 21mm 25 pk 4	REF PP004/1323 25mm 40 pk 4
REF PP004/1301 21mm 30 pk 4	REF PP004/1324 25mm 25/40 pk 4
REF PP004/1302 21mm 35 pk 4	REF PP004/1340 29mm 25 pk 4
REF PP004/1303 21mm 40 pk 4	REF PP004/1341 29mm 30 pk 4
REF PP004/1304 21mm 25/40 pk 4	REF PP004/1342 29mm 35 pk 4
REF PP004/1320 25mm 25 pk 4	REF PP004/1343 29mm 40 pk 4
REF PP004/1321 25mm 30 pk 4	REF PP004/1344 29mm 25/40 pk 4
REF PP004/1322 25mm 35 pk 4	

INDICATIONS FOR USE

For use by dental professionals during a root canal treatment for the placement of root canal sealer.

CONTRA INDICATIONS

None known.

INSTRUCTIONS FOR USE

1. Before cleaning the instrument, ensure there is no sign of damage, deformation or rust. Dispose of any instrument which is damaged.
2. It is essential that the instruments are cleaned and sterilised before first use and before reuse. Brush the instrument with a nylon brush. Ultrasonic sounds with a decontamination solution, rinse thoroughly and dry. Sterilise in a pouch at 134°C for 18 minutes.
3. Once sterilised, store in a sterilisation pouch such as Perfection Plus Self Seal pouch 002/5000.
4. Select the appropriate size for the treatment.
5. Connect to a contra angle hand piece. Select appropriate engine speed and torque. Test rotation before use to ensure the drill is securely fixed.
6. Irrigate and introduce the instrument into the canal, applying reciprocating and vertical filling movements.



WARNING!

It is recommended that PPE is worn when using this product, eye protection and gloves.
This product is sharp, avoid puncturing finger.
Dispose of product if it is damaged or has reduced efficacy.



STORAGE

Store in clean, dry conditions, protected from damp and corrosive products.



Batch code/Expiry Date

The life span is estimated at 3 years from the date of manufacturing but can vary depending on the use (cutting, conditions) on the frequency of use, and on the number of sterilisation cycles.

SPECIALLY FORMULATED FOR USE IN DENTISTRY

