

## Instructions for Use Peeso Reamers



### **IMPORTANT**

Read this 'Instructions for Use' document before handling. This product is for professional use only.

Peeso reamers are made from high grade stainless steel and are intended to be used to enlarge the coronal access. They are provided in a non-sterile state.

### **IDENTIFICATION**

Product Name	Peeso Reamers, Pack of 6
Company Name	Perfection Plus Ltd
Company Address	6 Westwood Court, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

### **PRODUCT CODE**

<b>REF</b>	PP004/4010 Peeso Reamers RA 1 - 0.70mm Pk 6
<b>REF</b>	PP004/4011 Peeso Reamers RA 2 - 0.90mm Pk 6
<b>REF</b>	PP004/4012 Peeso Reamers RA 3 - 1.10mm Pk 6
<b>REF</b>	PP004/4013 Peeso Reamers RA 4 - 1.30mm Pk 6
<b>REF</b>	PP004/4014 Peeso Reamers RA 5 - 1.50mm Pk 6
<b>REF</b>	PP004/4015 Peeso Reamers RA 6 - 1.70mm Pk 6
<b>REF</b>	PP004/4016 Peeso Reamers RA 1-6 Assorted Pk 6

### **INDICATIONS FOR USE**

For use by dental professionals during a root canal treatment to enlarge the coronal access.

### **CONTRA INDICATIONS**

None known.

### **INSTRUCTIONS FOR USE**

1. Before cleaning the instrument, ensure there is no sign of damage, deformation or rust. Dispose of any instrument which is damaged.
2. It is essential that the instruments are cleaned and sterilised before first use and before reuse. Brush the instrument with a nylon brush. Ultrasonic sounds with a decontamination solution, rinse thoroughly and dry. Sterilise in a pouch at 134°C for 18 minutes.
3. Once sterilised, store in a sterilisation pouch such as Perfection Plus Self Seal pouch 002/5000.
4. Select the appropriate size for the treatment.
5. Connect to a contra angle hand piece. Select appropriate engine speed and torque. Test rotation before use to ensure the drill is securely fixed.
6. Irrigate and introduce the instrument into the canal, applying reciprocating and vertical filling movements.



### **WARNING!**

It is recommended that PPE is worn when using this product, eye protection and gloves.  
This product is sharp, avoid puncturing finger.  
Dispose of product if it is damaged or has reduced efficacy.



### **STORAGE**

Store in clean, dry conditions, protected from damp and corrosive products.



### **Batch code/Expiry Date**

The life span is estimated at 3 years from the date of manufacturing but can vary depending on the use (cutting, conditions) on the frequency of use, and on the number of sterilisation cycles.

SPECIALLY FORMULATED FOR USE IN DENTISTRY

