

Instructions for Use PerfectEtch 2 x 5ml



IMPORTANT

Read this 'Instructions for Use' document before handling. This product is for use by a dental professional only.

IDENTIFICATION

PerfectEtch is formulated for the convenient preparation of enamel to retain resin based dental materials. It has optimum viscosity for easy extrusion with thixotropic 'non-drip' effect to remain where applied. Syringe presentation for ease of application and bright colour to ensure precise placement is achieved.

| | |
|-------------------|---|
| Product Name | PerfectEtch |
| Company Name | Perfection Plus Ltd |
| Company Address |  6, Westwood Court, Totton, Hants. SO40 3WX UK |
| Company Phone No. | +44 (0) 2380 866 677 |

PRODUCT CODE

REF PP010/0004 PerfectEtch 2 x 5ml syringes with 10 tips

INDICATIONS FOR USE

Treatment of enamel margins of class I, II, III and V composite restorations to prevent micro-leakage.
 Treatment of enamel surfaces of class IV composite restorations for retention.
 Treatment of occlusal surface of posterior teeth prior to application of fissure sealant resin.
 Treatment of enamel surfaces of teeth prior to the attachment of orthodontic appliances by means of resin adhesive.
 Treatment of the labial surface of teeth prior to application of resin-bonded ceramic or acrylic veneers.

INSTRUCTIONS FOR USE

Directions:
 If the enamel surface has not been freshly exposed by cavity preparation or tooth reduction, cleaning with Perfection Plus Oil Free Prophy Paste or pumice and water is recommended. Syringe the etch-gel directly on to the enamel surfaces to be treated. The gel can be spread using the Syringe tip or any other suitable instrument (small ball-ended plugger or brush). Leave etch gel in place for 30 seconds. Wash and dry using oil-free air. Enamel surface should have a dull frosted appearance. If not, repeat procedure as above.

WARNING!

The Etch-Gel contains 40% phosphoric acid. Corrosive. Causes burns. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. In case of accident or if you feel unwell, seek medical advice immediately (show label where possible).



PRECAUTIONS

Secure tip cap immediately after use. Do not force plunger; if blocked replace syringe tip.

STORAGE

Store in cool dry place (5-25°C)



Batch code/Expiry Date

The batch code given an open date of manufacture in month, year, day format with numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.
 The expiry date is shown in year, month format. Do not use the product after this date.

SPECIALLY FORMULATED FOR USE IN DENTISTRY

