

**Instructions for Use PerfectEtch 2 x 2ml****IMPORTANT**

Read this 'Instructions for Use' document before handling. This product is for use by a dental professional only.

**IDENTIFICATION**

PerfectEtch is formulated for the convenient preparation of enamel to retain resin based dental materials. It has optimum viscosity for easy extrusion with thixotropic 'non-drip' effect to remain where applied. Syringe presentation for ease of application and bright colour to ensure precise placement is achieved.

Product Name	PerfectEtch
Company Name	Perfection Plus Ltd
Company Address	6 Westwood Court, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

**PRODUCT CODE**

<b>REF</b> PP010/0086 PerfectEtch 2 x 2ml
---

**INDICATIONS FOR USE**

Treatment of enamel margins of class I, II, III and V composite restorations to prevent micro-leakage.

Treatment of enamel surfaces of class IV composite restorations for retention.

Treatment of occlusal surface of posterior teeth prior to application of fissure sealant resin.

Treatment of enamel surfaces of teeth prior to the attachment of orthodontic appliances by means of resin adhesive.

Treatment of the labial surface of teeth prior to application of resin-bonded ceramic or acrylic veneers.

**INSTRUCTIONS FOR USE**

1. If the enamel surface has not been freshly exposed by cavity preparation or tooth reduction, cleaning with Perfection Plus Oil Free Prophylaxis Paste or pumice and water is recommended.
2. Syringe the etch-gel directly on to the enamel surfaces to be treated. The gel can be spread using the Syringe tip or any other suitable instrument (small ball-ended plugger or brush).
3. Leave etch gel in place for 30 seconds.
4. Wash and dry using oil-free air.
5. Enamel surface should have a dull frosted appearance. If not, repeat procedure as above.

**WARNING!**

The Etch-Gel contains 40% phosphoric acid. Corrosive. Causes burns. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. In case of accident or if you feel unwell, seek medical advice immediately (show label where possible).

**PRECAUTIONS**

Secure tip cap immediately after use. Do not force plunger; if blocked, replace tip.

**STORAGE**

Store in a cool, dry place (5-25°C).

**Batch code/Expiry Date**

The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence. The expiry date is shown in year, month format. Do not use the product after this date.

**SPECIALLY FORMULATED FOR USE IN DENTISTRY**



www.perfectionplus.com



U123