

Instructions for Use PerfectEtch-E 12g



IMPORTANT

Read this 'Instructions for Use' document before handling. This product is for professional use only.

IDENTIFICATION

Product Name	PerfectEtch-E 12g
Company Name	Perfection Plus Ltd
Company Address	6 Westwood Court, Brunel Road, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

PRODUCT CODE

[REF] PP010/0090 12g Syringe and 10 disposable etch tips

INDICATIONS FOR USE

Phosphoric acid 37% etch gel is used during dental restoration procedures by professionals, for tooth surface etching (enamel and or dentin).

INSTRUCTIONS FOR USE

NOTE! Do not use if primary package is broken. It is recommended to use a dental dam during application of the product.

Preparation:

1. Prepare surface as always. The working area must be kept dry.
2. If the surface to be etched is not freshly exposed, cleaning with Perfection Plus Oil Free Prophylaxis Paste is recommended.
3. Attach applicator tip to the syringe.
4. Flow small amount of gel on mixing pad.

Etching:

5. Delicately push on plunger and apply layer of etch to all surfaces to be etched. Do not force plunger.
6. Leave etch gel in place for 15 seconds (dentine), 30 seconds (enamel).
7. Rinse with water and dry with air.
8. Verify etched surface is clean and dry.
9. Use an adhesive such as Connect-E, Connect+ or ConnectED.

WARNING!

NOTE! Do not use PerfectEtch-E if patient is allergic to product or to any of the ingredients.

Hazard and precautionary statements:

PerfectEtch-E contains phosphoric acid, which causes skin corrosion, severe skin burns and eye damage. IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower. Wash contaminated clothing before reuse. IF SWALLOWED: Rinse mouth. Do NOT induce vomiting. Subsequently, have the casualty drink 1 - 2 glasses of water immediately. Do not try to neutralize with bases, no charcoal. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER or doctor/physician.

Prevention:

Wash hands thoroughly after handling. Wear protective gloves / protective clothing / eye protection / face protection.



STORAGE

Keep tightly closed in dry well-ventilated place at 4-28°C. Protect from direct sunlight and heat sources.

Dispose of contents/container to as required by national regulatory requirements. Keep out of the reach of children!



Batch code/Expiry Date

The batch number should be quoted in all correspondence. See packaging for batch and expiry date. Do not use after expiry date.

SPECIALLY FORMULATED FOR USE IN DENTISTRY

