

Instructions For Use Restore



IMPORTANT

Read this 'Instructions for Use' document before handling. This product is for professional use only.

Restore is a universal composite suitable for anterior and posterior restorations. Excellent mechanical properties, high filler concentration and low shrinkage. Provides X-Ray opacity without impairing polishability. Colour shades are matched to the VITA shading system.

IDENTIFICATION

Product Name	Restore
Company Name	Perfection Plus Ltd
Company Address	6 Westwood Court, Brunel Road, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

PRODUCT CODE

REF PP010/1011 A1	REF PP010/1004 A3.5
REF PP010/1002 A2	REF PP010/1007 B2
REF PP010/1003 A3	REF PP010/1008 C3

INDICATIONS FOR USE

Tooth/Cavity Preparations. Follow usual procedures for tooth or cavity preparation in deep restorations or exposed dentin. Use of calcium hydroxide base such as PerfectaLINE is recommended.

INSTRUCTIONS FOR USE

1. Follow usual procedures for tooth or cavity preparation in deep restorations or exposed dentin.
2. Apply PerfectEtch to surface of teeth. Leave in place for 15 seconds if etching enamel or 10 seconds if treating dentine, according to instructions. Never leave etchant on the tooth for more than 60 seconds. Rinse thoroughly with water and dry tooth completely.
3. Apply a thick, uniform coating of Connect+ bond onto etched tooth surface according to manufacturer's Instructions for Use, and cure with light curing unit for 15-30 seconds.
4. Select the required shade of Restore, apply onto the prepared tooth in increments no larger than 2mm thick. Using a visible light-curing unit, with a minimum output of 600mW/cm², cure each incremental layer for at least 20 seconds.
5. Finish and polish using standard techniques. A composite surface sealant can also be used to seal microcracks and/or surface imperfections caused by the finishing procedures.

ADVERSE REACTION

Allergic or pre-existing skin, eye or respiratory conditions may be aggravated by exposure to this product.

CONTRA INDICATIONS

Perfection Plus Restore composite is contraindicated in patients with a history of severe allergic reaction to methacrylate resins.

WARNING!

Avoid contact of etchant with soft tissue, eyes or skin. Etchant contains phosphoric acid which is harmful. In case of contact with soft tissue, eyes or skin immediately flush with water for at least 15 minutes. Seek immediate medical attention.

Do not use in the presence of eugenol-based materials as complete setting may not occur.



STORAGE

Restore Light Cure Composite is formulated to be used at room temperature. See the outer package for expiry date. Do not freeze. Recommended storage conditions are between 2°C to 27°C, bringing to room temperature before using (23°C). Do not expose restorative materials to elevated temperatures or intense light. If using unit-dose tips, which are intended for single patient use, discard after each patient. If using syringes, always replace cap after each use.



BATCH CODE/EXPIRY DATE

The batch number should be quoted in all correspondence. See packaging for lot and expiry date. Do not use after expiry date.

SPECIALLY FORMULATED FOR USE IN DENTISTRY

