

Instructions for Use SealED**IMPORTANT**

Read this 'Instructions for Use' document before handling. This product is for professional use only.

IDENTIFICATION

Product Name	SealED
Company Name	Perfection Plus Ltd
Company Address	6 Westwood Court, Brunel Road, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

PRODUCT CODE

REF PP010/1200 Opaque	REF PP010/1210 Clear
------------------------------	-----------------------------

INDICATIONS FOR USE

For sealing pits and fissures on teeth showing no decayed lesions.

CONTRA INDICATIONS

SealED is contraindicated for patients with a history of severe allergic reactions to methacrylate resins.

INSTRUCTIONS FOR USE

1. Prepare the surface to be treated. The recommended method for preparing the occlusal surface is with an air polisher with sodium bicarbonate. If performing a prophylaxis, a prophy paste which contains oil or fluoride should not be used as it may leave a residue which inhibits bonding. We recommend Perfection Plus Oil Free Prophy Paste.
2. Rinse thoroughly to remove all residue.
3. Isolate the teeth to be treated with cotton rolls and/or rubber dam.
4. Dry the surfaces thoroughly with dry and oil-free air.
5. Apply PerfectEtch etchant gel following the instructions. Wait for 15 seconds: never leave the etchant on the tooth for longer than 60 seconds.
6. Rinse thoroughly with water, replace any wet cotton rolls if used. Avoid salivary contamination, if occurs repeat steps 4-6.
7. Dry the surfaces thoroughly with dry and oil-free air. The etched surface should be dull white, if not repeat steps 4-7.
8. Apply SealED using the black applicator tips to the pits and fissures. Brushes or another suitable instrument can be used to remove excess sealant.
9. Using a visible light curing unit with a minimum output of 600mW/cm², cure all coated surfaces for at least 20 seconds. Thicker or larger applications may require additional curing time.
10. Inspect the selected surfaces. If coverage is incomplete, apply additional sealant.
11. Check occlusion and adjust as required.

ADVERSE REACTION

Allergic or pre-existing skin, eye and respiratory conditions may be aggravated by exposure to this product.

WARNINGS

- Avoid contact of etchant with soft tissue, eyes and skin. Etchant contains phosphoric acid which is harmful. In case of contact with soft tissue, skin or eyes immediately flush with water for at least 15 minutes. Seek medical attention.
- Do not use in the presence of eugenol-based materials as complete setting may not occur.
- Applicator tips are disposable and intended for single patient use.
- Always replace cap after each use.

**STORAGE**

Recommended storage conditions are between 2°C to 27°C, bringing to room temperature before using. (23°C). Refrigeration is recommended. Do not expose to elevated temperatures or intense light. Do not freeze. Always replace cap after each use.

**BATCH CODE/EXPIRY DATE**

The batch number should be quoted in all correspondence. See packaging for lot and expiry date. Do not use after expiry date.

SPECIALLY FORMULATED FOR USE IN DENTISTRY

