

Instructions for Use Temporary Cavity Filling Material (TCFM)



IMPORTANT

Read this 'Instructions for Use' document before handling. This product is for use by a dental professional only.

IDENTIFICATION

A ready to use temporary filling material with a synthetic resin base. Adequate strength yet easily removed. Good adhesion to dentine and excellent marginal adaptation. Good sealing characteristic, easy to use consistency. It is not a permanent filling material and recommended use is up to a maximum of 4 weeks only.

Product Name	Temporary Cavity Filling Material (TCFM)
Company Name	Perfection Plus Ltd
Company Address	6 Westwood Court, Brunel Road, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

PRODUCT CODE

REF	PP010/0006 - 8g Tube
REF	PP010/0009 - 30g Jar

INDICATIONS FOR USE

Temporary filling of cavities. Temporary sealing of inlay preparations. Temporary sealing of endodontic treatments. Temporary fixing of post crowns.

INSTRUCTIONS FOR USE


Using a suitable filling instrument, place direct into cavity and adapt as necessary. It is not essential to dry the cavity prior to placement. Temporary filling material hardens by absorption of moisture to give an impermeable seal. It is recommended that masticatory pressure is not applied for approximately 1 hour.

For provisional fixing of post crowns spread a layer on inside of crown and seat in normal way.

NOTE: For tube presentation, remove cap from tube, reverse and pierce membrane fully. Use key provided to assist extrusion from tube.

STORAGE

Store in cool dry place (5 – 25°C). It is important that caps are tightly replaced after each use.

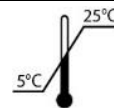

Batch code/Expiry Date

The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.

The expiry date is shown in year, month format. Do not use the product after this date.

SPECIALLY FORMULATED FOR USE IN DENTISTRY

