

EasiJect Sterile Needles

Instructions for Use

IMPORTANT

Read this 'Instructions for Use' document before use. This product is for professional use only.
SPECIALLY FORMULATED FOR USE IN DENTISTRY

IDENTIFICATION


Single Use Sterile dental injection needles for use in dental cartridge syringes for injection of dental local anaesthetic.

Product Name	EasiJect Sterile Needles
Distributed by	Perfection Plus Ltd
Company Address	6 Westwood Court, Brunel Road, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

PRODUCT CODE- REF

PP013/0200	EasiJect Sterile Needles 27Gx21mm Pk 100 Imperial
PP013/0203	EasiJect Sterile Needles 27Gx35mm Pk 100 Imperial
PP013/0206	EasiJect Sterile Needles 30Gx12mm Pk 100 Imperial
PP013/0208	EasiJect Sterile Needles 30Gx21mm Pk 100 Imperial
PP013/0209	EasiJect Sterile Needles 30Gx25mm Pk 100 Imperial
PP013/0300	EasiJect Sterile Needles 27Gx21mm Pk 100 Metric
PP013/0301	EasiJect Sterile Needles 27Gx25mm Pk 100 Metric
PP013/0303	EasiJect Sterile Needles 27Gx35mm Pk 100 Metric
PP013/0306	EasiJect Sterile Needles 30Gx12mm Pk 100 Metric
PP013/0307	EasiJect Sterile Needles 30Gx16mm Pk 100 Metric
PP013/0308	EasiJect Sterile Needles 30Gx21mm Pk 100 Metric
PP013/0309	EasiJect Sterile Needles 30Gx25mm Pk 100 Metric
PP013/0310	EasiJect Sterile Needles 30Gx30mm Pk 100 Metric

INSTRUCTIONS FOR USE

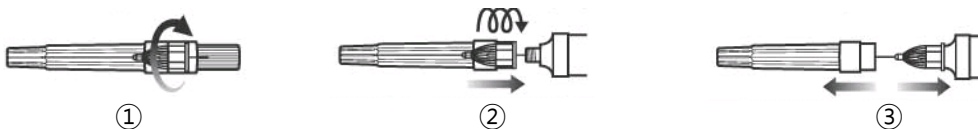

1. Break the hermetic seal (Twist cap to open – Do not bend or snap open).
2. Connect the needle on the dental cartridge syringe.
3. Remove the protective cap to use.

INDICATIONS FOR USE

- Use after checking the instruction for use.
- Do not use if the seal is already broken.
- Use right away after breaking the seal; The needle should remain inside the protective cap until use.
- Discard the needle tip by using the protective cap. Do not touch the needle tube to avoid cross infection after use.

CAUTION

Do not reuse to avoid infection and malfunction.

STERILE EO



Do not bend the metallic tube at the point where it joins the plastic. If needle tube is bent the needle can be broken and cause injury.



Do not bend the needle tube several times in use even small degrees of angle.



STORAGE

Store in cool, dry conditions. Avoid direct sunlight. Do not use if pack is damaged.



Distributed by: Perfection Plus Ltd
6 Westwood Court
Brunel Road
Totton
Hampshire
SO40 3WX, UK
www.perfectionplus.com



Obelis s.a
Bd General Wahis
53,1030 Brussels
Belgium
+ (32) 2 732-59-54



C-K DENTAL IND.
TEL : +82-32-679-1204 / 6
88-50, Sudo-ro, Bucheon-si,
Gyeonggi-do, Republic of Korea
ck-dental@hanmail.net

CE
1639

