

## RestorePLUS Flow

### Instructions for Use

#### **IMPORTANT**

Read this 'Instructions for Use' document before handling. This product is for use by dental professionals only.

#### **SPECIALLY FORMULATED FOR USE IN DENTISTRY**

#### **IDENTIFICATION**

An advanced radiopaque flowable composite for aesthetic restoration. The material has excellent handling based on flow properties (low flow type and high flow type) and excellent aesthetics by controlled light diffusion. Can be used for minimally invasive cavity restoration. Contains fluoride.

Product Name	RestorePLUS Flow
Distributed by	Perfection Plus Ltd
Company Address	6 Westwood Court, Brunel Road, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

#### **PRODUCT CODE – REF**

- PP010/1080 RestorePLUS Flow Syringe 2g A1 & 5 Tips
- PP010/1081 RestorePLUS Flow Syringe 2g A2 & 5 Tips
- PP010/1082 RestorePLUS Flow Syringe 2g A3 & 5 Tips
- PP010/1083 RestorePLUS Flow Syringe 2g A3.5 & 5 Tips
- PP010/1084 RestorePLUS Flow Syringe 2g A4 & 5 Tips

#### **INDICATIONS FOR USE**

- Anterior restorations such as class III and class V involving cervical caries, wedge-shaped defects.
- Small posterior restorations such as class I and class II without occlusal stress bearing.
- Deciduous restorations.
- Preventive restoration (pit & fissure sealant).
- Base / liner under restoration.
- Undercut blockout.
- Additional or marginal filling and recontouring after composite resin restoration.
- Colour adjustment on the surface of composite resin restoration and discoloured teeth.

#### **CONTRA INDICATIONS**

Do not use this product for any purposes other than specifically outlined under INDICATIONS FOR USE in these instructions for use

#### **INSTRUCTIONS FOR USE**

1. After cleaning the tooth surface and while the tooth is still moist check the shade.
2. Prepare the cavity in the conventional way, If the pulp is exposed or is close to the restoration, then a calcium hydroxide liner should be used.
3. Apply the bonding agent according to the manufacturer's directions. PerfectEtch & Connect+ is recommended for use in cavities.
4. Dispense the required amount of material from the syringe using a clean needle tip. Always replace the syringe cap after use.
5. After placing and contouring the restoration, cure with a light unit. Select the light curing time in the table below. Light cure the entire surface of the restoration. For deep cavities, the layering technique is recommended, Place in a layer of up to 2mm depth and cure with a dental light cure unit.

Light Curing Unit	LED curing unit	Halogen lamp curing unit
Light Curing Time	10 seconds	30 seconds
Wavelength	440-490 nm	400-500 nm`
Light Intensity	≥ 1000 mW/cm <sup>2</sup>	≥ 500 mW/cm <sup>2</sup>

Use a dental light-curing unit having specifications equivalent to those described above. When using the one having performance other than these recommendations, follow the instructions of each unit.

6. After curing, remove excess material with abrasive stones and points and then polish.

#### **STORAGE**

Keep away from direct sunlight and store at room temperature (1 - 30°C).



**WARNING!**

1. Do not expose patients or users known to be allergic to this type of material.
2. If any eruption or skin inflammation occurs on a patient when using this product, immediately discontinue use and have the patient seek medical advice.
3. Avoid contacting oral mucosa, skin, or eyes. In case of contact, rinse immediately with copious amounts of water and seek medical advice.
4. If a cavity is deep, protect with a pulp capping agent. (N.B. Do not use a eugenol containing liner).
5. Wear UV protective glasses when light curing.
6. This product has been formulated for use in dentistry and is intended for use by dental professionals only

**Caution**



Should a serious incident in relation to the product occur, it should be reported to the manufacturer and the competent authority.

**Batch code/Expiry Date**

The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence. The expiry date is shown in year, month format. Do not use the product after this date.



Advanced Healthcare Ltd  
2-4 Leavers State  
Chiddingstone Causeway  
Tonbridge  
Kent  
TN11 8JU, UK

Distributed by:  
Perfection Plus Ltd  
6 Westwood Court  
Brunel Road, Totton  
Hampshire  
SO40 3WX, UK  
[www.perfectionplus.com](http://www.perfectionplus.com)



Advena Ltd  
Tower Business  
Centre  
2nd Floor  
Tower Street  
Swatar, Malta



Perfection Plus EU Ltd  
The Black Church  
St Mary's Place.  
Dublin, Ireland,  
D07 P4AX  
Regulatory@perfectionplus.com

