

Impress-E Putty Instructions for Use

IMPORTANT

Read this 'Instructions for Use' document before handling. This product is for professional use only.
SPECIALLY FORMULATED FOR USE IN DENTISTRY

IDENTIFICATION

Dust-free material, however caution is advised when opening the bag and withdrawing the powder.

Product Name	Impress-E Putty Fast Set 2 x 300ml & 2 scoops
Company Name	Perfection Plus Ltd
Company Address	6 Westwood Court, Brunel Road, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

PRODUCT CODE - REF

PP010/1280 Impress-E Soft Putty Fast Set

PP010/1281 Impress-E Hard Putty Fast Set

INDICATIONS FOR USE

Preliminary impressions for the two-step putty/wash and one-step putty/wash techniques, base for the sandwich technique, reproduction of peripheries for full dentures, bite-taking. The impression can be electroplated.

INSTRUCTIONS FOR USE

Working and Set times for Product:

Working time: 2min 45s. Set time: 2min 15s. Total Work/Set time: 5 min.

1. Dry the areas to be reproduced and prevent further inflow of liquid into them.
2. Measure out two pastes in a ration 1:1 using the scoops.
3. Mix the two components vigorously to obtain a uniformly coloured mass.
4. Place the mixed mass on the impression tray.
5. Insert it into the mouth.

WARNINGS

The working and set time can be influenced by:

1. Variations in the temperature of the material, the environment and/or the operator's hands during mixing (higher temperatures reduce times whilst lower temperatures prolongs them)
2. Changes in the speed and/or force employed during mixing
3. Humidity
4. Contact of the silicone with latex rubber dam or gloves or with components of such material possibly present on the teeth during the impression procedure (vinyl gloves are recommended).



Do not invert lids of the containers and use the corresponding coloured scoop for each paste. Close the containers well after use. To eliminate traces of saliva or blood and guarantee the optimal adhesion of the wash impression to the preliminary impression, rinse under running water and dry well. The impression can be kept without any special devices. Cast the model in plaster at least 30 minutes after withdrawal from the mouth.

For disinfection, immerse the impression for 5 minutes in a solution of glutaraldehyde (concentration 0.5%-2%) or sodium hypochlorite (concentration 0.1%-1%). Rinse the impression under running water after disinfection.

Caution



Should a serious incident in relation to the product occur, it should be reported to the manufacturer and the competent authority.

STORAGE

Store in a cool, dry place.

**BATCH CODE/EXPIRY DATE**

See the print on the bag. Do not use the product after this date.



Perfection Plus Ltd
6 Westwood Court
Brunel Road
Hampshire
SO40 3WX, UK
www.perfectionplus.com



Perfection Plus EU Ltd
The Black Church
St Mary's Place.
Dublin, Ireland,
D07 P4AX
Regulatory@perfectionplus.com



Advena Ltd
Tower Business
Centre
2nd Floor Tower
Street
Swatar, Malta

