

PerfectaLINE Instructions for Use

IMPORTANT

Read this 'Instructions for Use' document before handling. This product is for use by dental professionals only.
SPECIALLY FORMULATED FOR USE IN DENTISTRY

IDENTIFICATION

PerfectaLINE is a rigid, self-setting, radiopaque composition that will not inhibit the polymerisation of acrylic and composite restorations.

Product Name	PerfectaLINE
Company Name	Perfection Plus Ltd
Company Address	6 Westwood Court, Brunel Road, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

PRODUCT CODE- REF

PP010/0025 PerfectaLINE Base Paste 13g & Catalyst Paste 11g

INDICATIONS FOR USE

Direct and indirect pulp capping.
Protective liner under dental filling materials, cements and other base materials.

CONTRAINDICATIONS

Hypersensitivity against sulphonamides or other components, respectively of the product. Acetone-containing products are detrimental to calcium hydroxide materials. Minimise contact of these products with calcium hydroxide liners.

INSTRUCTIONS FOR USE

Mixing:

Extrude equal volumes of the base and catalyst paste onto a mixing pad. Mix immediately using a spatula until a uniform colour is achieved. **DO NOT OVER SPATULATE.** To provide maximum working time, complete mixing of the two pastes within 10 seconds.

Application:

- Quickly apply the mixed material into the cavity for efficient placement before setting starts. The cavity preparation should be dry.
- Avoid touching the margins of the cavity. Avoid placing a large bulk of material.
- The mixed material will set in 2 to 4 minutes in the mouth.
- Remove any excess set material for retention areas or margins with a sharp spoon excavator or bur.

Increased humidity and elevated temperature will reduce available work time and set time. Conversely, decreased humidity and lower temperature will increase available work time and set time.

WARNINGS

May cause allergic reactions in persons susceptible to sulphonamides. Contains calcium hydroxide. Avoid contact with skin and eyes. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. In case of accident or if you feel unwell, seek medical advice immediately (show label where possible).

Base paste contains 1,3-Butylene Glycol Disalicylate. Irritating to skin and eyes. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. In case of contact with skin, wash immediately with plenty of soap and water.

Caution



Should a serious incident in relation to the product occur, it should be reported to the manufacturer and the competent authority.

STORAGE

Store in a cool, dry place (5-25°C). It is important to keep tubes tightly sealed when not in use. Separation of solid and liquid phases of the base plate is accelerated at elevated temperatures. When stored at reduced temperatures, allow the material to return to room temperature before using.

Batch code/Expiry Date

The batch code gives an open date of manufacture in month, year, and day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence. The expiry date is shown in year, month format. Do not use the product after this date.



Perfection Plus Ltd
6 Westwood Court
Brunel Road
Hampshire
SO40 3WX, UK
www.perfectionplus.com



Perfection Plus EU Ltd
The Black Church
St Mary's Place.
Dublin, Ireland,
D07 P4AX
Regulatory@perfectionplus.com



Advena Ltd
Tower Business
Centre
2nd Floor Tower
Street
Swatar, Malta

