

Perfection Plus Retraction Cord

Instructions for Use

IMPORTANT

Read this 'Instructions for Use' document before handling. This product is for professional use only.

IDENTIFICATION

Product Name	Perfection Plus Retraction Cord
Company Name	Perfection Plus Ltd
Company Address	6 Westwood Court, Brunel Road, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

PRODUCT CODE - [REF]

PP010/0040 – Size 0 length 300cm

PP010/0041 – Size 1 length 300cm

PP010/0042 – Size 2 length 300cm

COMPOSITION

Pure Combed Cotton

Bleached with H2O2, no chlorine.

Free of any optical brightener

European Pharmacopoeia Standards

INDICATIONS FOR USE

Retraction of the gingiva prior to impression.

INSTRUCTIONS FOR USE

The retraction produced by the cord is a mechanical retraction. It temporarily opens the Sulcus to enable an impression or dental treatment to be undertaken. one tooth is to be treated at a time.

Pull the cord out of the bottle and cut the required length. Cut longer sections than required so the ends overlap when wound around the tooth, making removal easier.

Place the cord in the sulcus using a blunt metal instrument. Start with the interproximal space where the Sulcus is usually deepest. To obtain greater retraction, once the cord has been inserted it may be impregnated with a solution for retraction, leave in situ for the time indicated in the Solution Manufacturer Instructions for use. The cord should be removed just before taking the impression. Slightly moisten the cord to make removal easier and less painful.

STORAGE

Store in dry conditions.



WARNING

After taking the impression the gingival sulcus should be inspected to make sure that the epithelial attachment has not been torn and no fragment of the retraction cord left in the gingival crevice. The cord should only be used on healthy tissues free from injury, bleeding, or infection. Should a serious incident in relation to the product occur, it should be reported to the manufacturer and the competent authority.



BATCH CODE/EXPIRY DATE

The batch number should be quoted in all correspondence. The batch code gives an open date of manufacture month, year, and day format, with a numerical suffix to uniquely identify the batch of material. Do not use after expiry date.



Perfection Plus Ltd
6 Westwood Court
Brunel Road
Totton
Hampshire
SO40 3WX, UK
www.perfectionplus.com



Perfection Plus EU Ltd
The Black Church
St Mary's Place
Dublin, Ireland
D07 P4AX
regulatory@perfectionplus.com



Advena Ltd
Tower Business
Centre
2nd Floor Tower
Street
Swatar, Malta

