

## RootPLUS

### Instructions for Use

#### **IMPORTANT**

Read this 'Instructions for Use' document before handling. This product is for use by a dental professional only.

#### **SPECIALLY FORMULATED FOR USE IN DENTISTRY**

#### **IDENTIFICATION**

A zinc paste-paste zinc oxide eugenol cement for use with obturation points. The 2 pastes mix easily to a smooth creamy consistency with long working and setting time. The cement has moderate flow characteristics, is radiopaque and is low in film thickness (about 6µm).

Product Name	RootPLUS
Company Name	Perfection Plus Ltd
Company Address	6 Westwood Court, Brunel Road, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

#### **PRODUCT CODE - REF**

PP004/0200 RootPLUS Base Paste 8g & Catalyst Paste 3.5g

#### **INDICATIONS FOR USE**

Permanent obturation of the root canal space with the aid of obturation points.

#### **CONTRAINDICATIONS**

1. Zinc Oxide Eugenol based sealants in general may produce periapical inflammation in the first few days after placement. Such inflammation should subside naturally as healing progresses.
2. There are no known systemic side-effects to date. However, some people may exhibit sensitivity to Eugenol. If such reactions are experienced, discontinue use of product and seek medical assistance.

#### **INSTRUCTIONS FOR USE**

Base Paste : Catalyst Paste Ratio	2cm : 2cm 0.33g : 0.15g
Mixing time	20 seconds
Working Time On Pad (at 23±1°C and 50±10% RH)	10-25 minutes
Setting Time In Mouth (at 37±1°C)	Less than 60 minutes

- Squeeze equal lengths of base paste and catalyst paste on pad and mix thoroughly with flat bladed spatula to a creamy consistency of uniform colour.
- Coat the obturation point with cement.
- Spread a layer of cement over all inside surfaces of root canal using an appropriate carrier (e.g. endodontic file, gutta percha or paper point).
- Seat obturation point slowly using normal procedures.

#### **STORAGE**

Store in a cool dry place (5-25°C), Close tubes immediately after use.



**Batch code/Expiry Date**

The batch code gives an open date of manufacture in month, year, and day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence. The expiry date is shown in year, month format. Do not use the product after this date.

**Caution**

Should a serious incident in relation to the product occur, it should be reported to the manufacturer and the competent authority.



Perfection Plus Ltd  
6 Westwood Court  
Brunel Road  
Hampshire  
SO40 3WX, UK  
[www.perfectionplus.com](http://www.perfectionplus.com)



Perfection Plus EU Ltd  
The Black Church  
St Mary's Place.  
Dublin, Ireland,  
D07 P4AX  
[Regulatory@perfectionplus.com](mailto:Regulatory@perfectionplus.com)



Advena Ltd  
Tower Business  
Centre  
2nd Floor Tower  
Street  
Swatar, Malta



Perfection Plus Ltd  
6 Westwood Court  
Brunel Road  
Totton  
Hampshire  
SO40 3WX, UK  
[www.perfectionplus.com](http://www.perfectionplus.com)



Perfection Plus EU Ltd  
The Black Church  
St. Marys Place  
Dublin, Ireland  
D07 94AX  
[Regulatory@perfectionplus.com](mailto:Regulatory@perfectionplus.com)



Advena Ltd  
Tower Business  
Centre  
2nd Floor Tower  
Street  
Swatar, Malta

