

PoliSTRIP Instructions for Use

IMPORTANT


Read this 'Instructions for Use' document before handling. This product is for use by a dental professional only. PoliSTRIP are used by the Dental professional for the finishing and polishing of composite restorations.

Product Name	PoliSTRIP
Company Name	Perfection Plus Ltd
Company Address	6 Westwood Court, Brunel Road, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

PRODUCT CODE – REF

PP010/1860 4mmx150mm Ultra Fine/Fine pk 50
PP010/1861 4mmx150mm Medium/Coarse pk 50

STERILISATION

The strips do not require sterilisation. The strips should be kept in their original packaging at room temperature and protected against dust and moisture until used. 

STORAGE

The strips should be kept in hygienic stands, dishes or other suitable containers and they must be protected against dust, moisture, and contamination during storage. If the strips are not used right away, it is advisable to keep them in their original packaging.

PROPER USE

- Products that are deformed should not be used and must be disposed of.
- Use the strips in the correct order, incorrect procedures can result in a reduction in the polishing quality.
- Wear a face mask to prevent biological transfer.
- Incorrect use produces poor results and increases the risk. These products must only be used by qualified staff.

PRESSURE

- Excessive pressure must be always avoided.
- Excessive pressure may cause heat build-up which may damage the strip.

Excessive pressure may cause overheating, which injures the pulp. In extreme cases, the instrument may even fracture

INSTRUCTIONS

1. Insert the central section (the area without abrasive) of the finishing strip between contact points.
2. Position the selected portion of the strip over the composite surface to be finished.
3. Firmly grasp both ends of the strip and draw the abrasive over the composite in a vigorous, back and forth motion
4. Repeat procedure using another grade of the strip.
5. Discard strip.
6. Repeat steps 1-5 with Polishing strips

DISPOSE OF WORN INSTRUMENTS

- Fractured and incorrectly shaped strips can cause injury.

SAFETY PRECAUTIONS

These dental instruments were developed and manufactured for their specific dental surgical application. Incorrect use may harm tissue, cause premature wear, destroy the instruments, and endanger the operator, patient or third parties.

 Stoddard Manufacturing
Company Ltd
Black Horse Road
Letchworth
Herts, SG6 1HB, UK

Distributed by:
Perfection Plus Ltd
6 Westwood Court
Brunel Road, Totton
Hampshire
SO40 3WX, UK
www.perfectionplus.com



Stoddard ApS
Moellegade 32
8000 Aarhus C
Denmark



Perfection Plus EU Ltd
The Black Church
St Mary's Place.
Dublin, Ireland,
D07 P4AX
Regulatory@perfectionplus.com

