

Fix II
Instructions for Use

IMPORTANT

Read this 'Instructions for Use' document before handling. This product is for use by a dental professional only.
SPECIALLY FORMULATED FOR USE IN DENTISTRY

IDENTIFICATION

Product Name	Fix 11 Tray Adhesive 2 x14ml
Company Name	Perfection Plus Ltd
Company Address	6 Westwood Court, Brunel Road, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

PRODUCT CODE - REF

PP010/2010 Tray Adhesive 2x14ml

INDICATIONS FOR USE

Bonds alginate and hydrocolloid impression materials to metal or plastic trays

INSTRUCTIONS FOR USE

1. Brush a thin layer of Fix II Tray Adhesive on the inside surface of the tray and slightly over the peripheral border. Allow to dry for approximately 3 minutes.
2. Place the impression material in the tray while the Fix II Tray Adhesive is still tacky
3. Take the impression as directed for the impression material.
4. After the impression material has set, it will remain attached to the tray, thus helping to achieve a more accurate impression.
5. Clean the tray with isopropyl alcohol.

CONTRAINDICATIONS FOR USE Not recommended for use with elastomeric impression materials.

WARNING!

Avoid contact of with eyes or skin. In case of contact with eyes, flush with copious amounts of water for at least 15 minutes. Seek immediate medical attention.

Vapour is harmful. Use only in a well-ventilated area. If inhaled, move to fresh air.

Do not apply directly to teeth, skin, or oral tissues.

Fix II tray adhesive is flammable. Keep away from heat, spark, or open flame.

Review MSDS sheet before use and use only as directed.



Caution



Should a serious incident in relation to the product occur, it should be reported to the manufacturer and the competent authority.


STORAGE

Recommended storage conditions are between 2°C to 27°C, bringing to room temperature (22°C) before using. See outer package for expiry date. Do not expose to elevated temperatures or sources of ignition. Keep containers tightly closed when not in use.




Batch code/Expiry Date

See pot for Lot number and expiration date.

 Dental Technologies
 6901 North Hamlin Avenue
 Lincolnwood
 Illinois 60712, USA
www.dentaltech.com

Distributed by:
 Perfection Plus Ltd
 6 Westwood Court
 Brunel Road, Totton
 Hampshire
 SO40 3WX, UK
www.perfectionplus.com

 **EC REP** Emergo Europe
 Princsessgracht 20
 2514AP The
 Hague
 Netherlands

 Perfection Plus EU Ltd
 The Black Church
 St Mary's Place.
 Dublin, Ireland,
 D07 P4AX
Regulatory@perfectionplus.com

