



Titanium and Gold Screw Posts

Instructions for Use

IMPORTANT

Read this 'Instructions for Use' document before handling. This product is for use by dental professionals only.
SPECIALLY FORMULATED FOR USE IN DENTISTRY

IDENTIFICATION

Perfection Plus Screw Posts are used to give retention for reconstruction of non-vital teeth, where the root canal has been endodontically treated. They are supplied non-sterile and should be sterilised before use. They are single use instruments.



Product Name	Titanium and Gold Screw Posts
Company Name	Perfection Plus Ltd
Company Address	6 Westwood Court, Brunel Road, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

PRODUCT CODE – REF

012/3000	Gold Screw Post Kit 60	012/4000	Titanium Screw Post Kit 60
012/3001	Gold Screw Post Kit 240	012/4001	Titanium Screw Post Short 1
012/3002	Gold Screw Post Short 1	012/4002	Titanium Screw Post Short 2
012/3003	Gold Screw Post Short 2	012/4003	Titanium Screw Post Short 3
012/3004	Gold Screw Post Short 3	012/4004	Titanium Screw Post Short 4
012/3005	Gold Screw Post Short 4	012/4005	Titanium Screw Post Short 5
012/3006	Gold Screw Post Short 5	012/4006	Titanium Screw Post Short 6
012/3007	Gold Screw Post Short 6	012/4007	Titanium Screw Post Medium 1
012/3008	Gold Screw Post Medium 1	012/4008	Titanium Screw Post Medium 2
012/3009	Gold Screw Post Medium 2	012/4009	Titanium Screw Post Medium 3
012/3010	Gold Screw Post Medium 3	012/4010	Titanium Screw Post Medium 4
012/3011	Gold Screw Post Medium 4	012/4011	Titanium Screw Post Medium 5
012/3012	Gold Screw Post Medium 5	012/4012	Titanium Screw Post Medium 6
012/3013	Gold Screw Post Medium 6	012/4013	Titanium Screw Post Long 1
012/3014	Gold Screw Post Large 1	012/4014	Titanium Screw Post Long 2
012/3015	Gold Screw Post Large 2	012/4015	Titanium Screw Post Long 3
012/3016	Gold Screw Post Large 3	012/4016	Titanium Screw Post Long 4
012/3017	Gold Screw Post Large 4	012/4017	Titanium Screw Post Long 5
012/3018	Gold Screw Post Large 5	012/4018	Titanium Screw Post Long 6
012/3019	Gold Screw Post Large 6	012/4019	Titanium Screw Post X-Long 4
012/3020	Gold Screw Post X-Large 4	012/4020	Titanium Screw Post X-Long 6
012/3021	Gold Screw Post X-Large 6		

INDICATIONS FOR USE

CONTRAINDICATIONS

Do not use this product for a patient who indicates sensitisation or allergic reaction.

PRECAUTIONS FOR USE

The gold-plated screw posts should not be shortened or tampered with in any way that might take away part of the protective gold layer as this can cause the post to corrode. Titanium posts are not affected by this.

INSTRUCTIONS FOR USE

1. Screw posts should be sterilised before use (with a validated autoclave cycle at 134°C for 18 minutes minimum).
2. It is of utmost importance that the root canal has been prepared with a reamer of appropriate size and shape corresponding to the post to be used.
3. When using reamers, start with a reamer number smaller than the diameter number desired and proceed from reamer number to reamer number step by step until desired reamer can be used. Reamers are numbered 1-6.
4. Reamers should be used at moderate r.p.m. speed and with a careful up-and-down movement only. If these precautions are not taken the reamer can get stuck and/or break off in the root canal.
5. Never use force when inserting post as this can cause root fracture.
6. Use appropriate cement to increase retention of the post.
7. Screw posts should be used "Lege Artis" and with professional judgement.
8. Screw posts should be sterilised before use (with a validated autoclave cycle at 134°C for 18 minutes minimum).

STERILISATION

Sterilise posts before use at 134°C in a steam autoclave for a minimum of 18 minutes.



CAUTION

Should a serious incident in relation to the product occur, it should be reported to the manufacturer and the competent authority.



LOT NUMBER

The lot number is shown as LOT on each pack. Please quote this in all correspondence.



Harald Nordin S.A
Z.I. La Foge
CH-1816 Chailly s/Montreux
Switzerland
Tél: +41 21 964 71 71
E-mail: info@nordin-dental.com



Perfection Plus Ltd
6 Westwood Court
Brunel Road, Totton
Hampshire
SO40 3WX, UK
www.perfectionplus.com



WMDE B.V.
Bergerweg 18
6085 AT Horn
The Netherlands



Perfection Plus EU Ltd
The Black Church
St Mary's Place.
Dublin, Ireland,
D07 P4AX
Regulatory@perfectionplus.com



1639