

Fix II
Instructions for Use

IMPORTANT

Read this 'Instructions for Use' document before handling. This product is for use by a dental professional only.

SPECIALLY FORMULATED FOR USE IN DENTISTRY

IDENTIFICATION

Product Name	Fix II Tray Adhesive 2 x 14ml
Company Name	Perfection Plus Ltd
Company Address	6 Westwood Court, Brunel Road, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

PRODUCT CODE - REF

PP010/2010 Tray Adhesive 2 x 14ml

INDICATIONS FOR USE

Bonds alginate and hydrocolloid impression materials to metal or plastic trays

INSTRUCTIONS FOR USE

1. Brush a thin layer of Fix II Tray Adhesive on the inside surface of the tray and slightly over the peripheral border. Allow to dry for approximately 3 minutes.
2. Place the impression material in the tray while the Fix II Tray Adhesive is still tacky
3. Take the impression as directed for the impression material.
4. After the impression material has set, it will remain attached to the tray, thus helping to achieve a more accurate impression.
5. Clean the tray with isopropyl alcohol.

CONTRAINDICATIONS FOR USE

Not recommended for use with elastomeric impression materials.

WARNING!

Avoid contact of with eyes or skin. In case of contact with eyes, flush with copious amounts of water for at least 15 minutes. Seek immediate medical attention.

Vapour is harmful. Use only in a well-ventilated area. If inhaled, move to fresh air.

Do not apply directly to teeth, skin, or oral tissues.

Fix II tray adhesive is flammable. Keep away from heat, spark, or open flame.

Review MSDS sheet before use and use only as directed.



Caution



Should a serious incident in relation to the product occur, it should be reported to the manufacturer and the competent authority.

STORAGE

Recommended storage conditions are between 2°C to 27°C, bringing to room temperature (22°C) before using. See outer package for expiry date. Do not expose to elevated temperatures or sources of ignition. Keep containers tightly closed when not in use.




Batch code/Expiry Date

See pot for Lot number and expiration date.

Distributed by: Perfection Plus Ltd, 6 Westwood Court, Brunel Road, Totton, Hampshire, SO40 3WX

 Dental Technologies
6901 North Hamlin
Avenue
Lincolnwood
Illinois 60712, USA
www.dentaltech.com

 Perfection Plus Ltd
6 Westwood Court
Brunel Road, Totton
Hampshire
SO40 3WX, UK
www.perfectionplus.com
Regulatory@perfectionplus.com

 Emurgo Europe
Westervoortsedijk 60
6827 AT, Arnhem
The Netherlands

 Perfection Plus EU Ltd
The Black Church
St Mary's Place.
Dublin, Ireland,
D07 P4AX
Regulatory@perfectionplus.com

