

PerfectEtch-E

Instructions for Use

IMPORTANT

Read this 'Instructions for Use' document before handling. This product is for use by dental professionals only.

SPECIALLY FORMULATED FOR USE IN DENTISTRY

IDENTIFICATION

Thixotropic 37% phosphoric acid etch gel with optimum level of viscosity. Syringe presentation with special tips allows placement of the etching gel in any desired location.

Product Name	PerfectEtch-E
Company Name	Perfection Plus Ltd
Company Address	6 Westwood Court, Brunel Road, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

PRODUCT CODE - REF

010/0090 12g Syringe and 10 disposable etch tips.

010/0092 60g Syringe and 50 disposable etch tips

INDICATIONS FOR USE

Thixotropic 37% phosphoric acid etch gel is used during dental restoration procedures by professionals, for tooth surface etching (enamel and or dentin).

CONTRA INDICATIONS

Do not use if patient is allergic to product or to any of the ingredients.

INSTRUCTIONS FOR USE

NOTE! Do not use if primary package is broken. It is recommended to use a dental dam during application of the product.

Preparation:

1. Prepare surface as usual. The working area must be kept dry.
2. If the surface to be etched is not freshly exposed, cleaning with Perfection Plus Oil Free Prophylaxis Paste is recommended.
3. Attach applicator tip to the syringe.
4. Flow small amount of gel on mixing pad.


Etching:

5. Delicately push on plunger and apply layer of etch to all surfaces to be etched. Do not force plunger.
6. Leave etch gel in place for 15 seconds (dentine), 30 seconds (enamel).
7. Rinse with water and dry with air.
8. Verify etched surface is clean and dry.
9. Use an adhesive such as Connect-E or Connect+.

STORAGE

Keep container tightly closed in dry well-ventilated place between 4-28°C. Protect from direct sunlight and heat sources.

Dispose of contents/container as required by national regulatory requirements. Keep out of the reach of children!

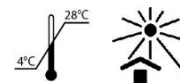

WARNING!

Hazard and precautionary statements:

PerfectEtch-E contains phosphoric acid, which causes skin corrosion, severe skin burns and eye damage.

- IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower. Wash contaminated clothing before reuse.
- IF SWALLOWED: Rinse mouth. Do NOT induce vomiting. Subsequently, have the casualty drink 1 - 2 glasses of water immediately. Do not try to neutralize with bases, no charcoal.
- IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses if present and easy to do. Continue rinsing. Immediately call a POISON CENTER or doctor/physician.



Prevention:

Wash hands thoroughly after handling. Wear protective gloves / protective clothing / eye protection / face protection.

Caution



Should a serious incident in relation to the product occur, it should be reported to the manufacturer and the competent authority.

Batch code/Expiry Date

See packaging for batch and expiry date. Do not use after expiry date. Please quote this batch number in all correspondence.



Perfection Plus Ltd
6 Westwood Court
Brunel Road
Hampshire
SO40 3WX, UK
www.perfectionplus.com



Perfection Plus EU Ltd
The Black Church
St Mary's Place.
Dublin, Ireland,
D07 P4AX
Regulatory@perfectionplus.com



Advena Ltd
Tower Business
Centre
2nd Floor Tower
Street
Swatar, Malta

