

Lip and Cheek Retractor


Instructions for Use

IMPORTANT

Read this 'Instructions for Use' document before handling. This product is for professional use only.
SPECIALLY FORMULATED FOR USE IN DENTISTRY

IDENTIFICATION

Transparent blue plastic lip and cheek retractor with or without a handle.

Product Name	Lip and Cheek Retractor
Company Name	Perfection Plus Ltd
Company Address	 6 Westwood Court, Brunel Road, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

PRODUCT CODE - REF

PP011/0090 - Lip and Cheek Retractor Adult pk 2
 PP011/0091 - Lip and Cheek Retractor Child pk 2
 PP011/0092 - Lip and Cheek Retractor Adult with handle pk 2
 PP011/0093 - Lip and Cheek Retractor Child with handle pk 2

INDICATIONS FOR USE

Used to hold the lips and cheeks open during a dental examination or treatment.

FUNCTION CHCK PRIOR TO USE

Prior to use, check for any possible damage or distortion.

CLEANING INSTRUCTION

After cleaning with a suitable CE marked disinfectant solution, rinse thoroughly under running water. If using a washer disinfector do not exceed 93°C.

AUTOCLAVE INSTRUCTION

Autoclave using steam sterilisation for 15 minutes at 121°C.

STORAGE

No special requirements.

BATCH CODE/EXPIRY DATE

The batch number should be quoted in all correspondence. The device can be used up to a maximum of 50 sterilisation cycles.

CAUTION



Should a serious incident in relation to the product occur, it should be reported to the manufacturer and the competent authority.

