

Mouth Prop

Instructions for Use

IMPORTANT

Read this 'Instructions for Use' document before handling. This product is for professional use only.
SPECIALLY FORMULATED FOR USE IN DENTISTRY

IDENTIFICATION

Mouth Props are available in 3 sizes and are used during a dental procedure to hold a patients mouth open.

Product Name	Mouth Prop
Company Name	Perfection Plus Ltd
Company Address	6 Westwood Court, Brunel Road, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

PRODUCT CODE - REF

011/0250 – Mouth Prop Small Yellow
 011/0251 – Mouth Prop Medium Green
 011/0252 – Mouth Prop Large Purple
 011/0260 – Mouth Prop with Chain Small Yellow
 011/0261 – Mouth Prop with Chain Medium Blue
 011/0262 – Mouth Prop with Chain Large Black

INTENDED USER

Device use is restricted to appropriately qualified dental professionals. No users outside the field of dentistry have been identified.

INTENDED PATIENT POPULATION

The devices are intended for use on patients of any age as required, subject to user opinion where appropriate.

INDICATIONS FOR USE

The Mouth Prop is placed between the patient's teeth to hold the mouth open. The product has a supporting structure that for ces the mandible open during oral surgery, preventing fatigue in the patient.

FUNCTION CHECK PRIOR TO USE

Prior to use, check for any possible damage or distortion.

CLEANING AND STERILISATION INSTRUCTION

After use, rise with clean running water before soaking in a CE marked multi-enzyme cleaning solution, following the manufacturer's instructions. Rinse thoroughly and autoclave for 20 minutes at 121°C.

STORAGE

No special requirements.

BATCH CODE/EXPIRY DATE

The batch number should be quoted in all correspondence.

Caution



Should a serious incident in relation to the product occur, it should be reported to the manufacturer and the competent authority.



Perfection Plus Ltd
 6 Westwood Court
 Brunel Road
 Hampshire
 SO40 3WX, UK
www.perfectionplus.com



Perfection Plus EU Ltd
 The Black Church
 St Mary's Place.
 Dublin, Ireland,
 D07 P4AX
Regulatory@perfectionplus.com



Advena Ltd
 Tower Business
 Centre
 2nd Floor Tower
 Street
 Swatar, Malta

