

Amalgam Carrier and Replacement Tips

Instructions for Use

IMPORTANT

Read this 'Instructions for Use' document before handling. This product is for professional use only.
SPECIALLY FORMULATED FOR USE IN DENTISTRY

IDENTIFICATION

Plastic Amalgam carrier for dental amalgam and replacement tips.

Product Name	Amalgam Carrier Plastic
Company Name	Perfection Plus Ltd
Company Address	6 Westwood Court, Brunel Road, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

PRODUCT CODE - [REF]

PP011/0040 – Amalgam Carrier Plastic Straight
PP011/0041 – Amalgam Carrier Plastic RA
PP011/0041 – Amalgam Carrier Plastic Curved

PP011/0043 - Replacement Tip Amalgam Carrier Straight
PP011/0044 - Replacement Tip Amalgam Carrier Curved
PP011/0045- Replacement Tip Amalgam Carrier RA

INDICATIONS FOR USE

Designed for the application of dental amalgam.

INSTRUCTIONS FOR USE

Push the tip of the carrier into the amalgam compound in order to fill the hollow cylinder of the carrier tip. Press the carrier plunger to deposit the amalgam compound on to the area being treated.

AUTOCLAVE INSTRUCTION

Dismantle the carrier by unscrewing the tip before cleaning thoroughly. The plastic is resistant to disinfectant chemicals but will melt at 166°C. Avoid contact with other metals during the autoclave process. Autoclave for 20 minutes at 121°C at 1 atm or at 134°C for 8 minutes at 2 atm.

STORAGE

Store in a clean dry place, away from direct heat, dust and atmospheric agents.

BATCH CODE/EXPIRY DATE

The batch number should be quoted in all correspondence. The device can be used up to a maximum of 100 sterilisation cycles.

Caution



Should a serious incident in relation to the product occur, it should be reported to the manufacturer and the competent authority.



Perfection Plus Ltd
6 Westwood Court
Brunel Road
Totton
Hampshire
SO40 3WX, UK
www.perfectionplus.com



Perfection Plus EU Ltd
The Black Church
St Mary's Place
Dublin, Ireland
DO7 P4AX
regulatory@perfectionplus.com



Advena Ltd
Tower Business
Centre
2nd Floor Tower
Street
Swatar, Malta

