

ConnectED

Instructions for Use

IMPORTANT

Read this 'Instructions for Use' document before handling. This product is for use by dental professionals only.
SPECIALLY FORMULATED FOR USE IN DENTISTRY

IDENTIFICATION

A HEMA-free all in one adhesive to etch, prime and bond in one step. It offers excellent bond strength to enamel and dentine. Bond durability and biocompatibility are outstanding.

Product Name	ConnectED
Company Name	Perfection Plus Ltd
Company Address	6 Westwood Court, Brunel Road, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

PRODUCT CODE - REF

PP010/0120 - ConnectED Kit

PP010/0121 - ConnectED 6ml bottle – Triple bottle pack

PP010/0122 - ConnectED 6 bottle

INDICATIONS FOR USE

Suitable for use with composites and compomers.

Side Effects:

Isolated cases of contact allergies with products of similar composition have been reported. To avoid possible pulpal reactions, protect the exposed dentine in the pulpal area with an appropriate lining material (preferably with a calcium hydroxide cavity liner such as PerfectaLINE).

Interactions:

Avoid cavity liners which might inhibit polymerisation due to their components, e.g. phenolic substances such as zinc oxide eugenol.

CONTRAINDICATIONS

Exposed pulp, pulpitis, and known allergy to methacrylate resin.

INSTRUCTIONS FOR USE

1. Clean the tooth surface and prepare the cavity following conventional methods. Isolate with rubber dam if appropriate.
2. Apply calcium hydroxide cavity liner (PerfectaLINE) in deep cavities where the remaining dentine is thin.
3. Apply a thin layer of adhesive and leave undisturbed for 10 seconds. Dry gently with oil-free air for 5 seconds.
4. Light cure with Perfect+ LED05 curing light for 5 seconds. Cavity is now ready for filling with composite or compomer restorative.

WARNING!

Close bottle after each use. Keep material out of the reach of children. In case of direct contact with oral mucosa, rinse with plenty of water. Harmful if swallowed. Irritating to eyes and skin. May cause sensitisation by skin contact. In case of contact with eyes, rinse immediately with water and seek medical advice. After contact with skin, wash immediately with plenty of soap and water. Wear suitable protective clothing, gloves and eye/face protection. Highly flammable liquid.



Caution



Should a serious incident in relation to the product occur, it should be reported to the manufacturer and the competent authority.

STORAGE

Store in a refrigerator (1-10°C).

Caution: Do not expose to direct sunlight or other heat sources. Do not freeze.



Batch code/Expiry Date

The batch number should be quoted in all correspondence. The batch code gives an open date of manufacture month, year and day format, with a numerical suffix to uniquely identify the batch of material. Do not use after expiry date shown.



Perfection Plus Ltd
6 Westwood Court
Brunel Road
Hampshire
SO40 3WX, UK
www.perfectionplus.com



Perfection Plus EU Ltd
The Black Church
St Mary's Place.
Dublin, Ireland,
D07 P4AX
Regulatory@perfectionplus.com



Advena Ltd
Tower Business
Centre
2nd Floor Tower
Street
Swatar, Malta

