

## EasiJect Sterile Needles

### Instructions for Use

Single Use sterile dental needles.

#### Intended Purpose

Sterile dental injection needles for single use which are used in dental cartridge syringe for injection of dental local anaesthetics.



Product Name	EasiJect Sterile Needles
Distributed by	Perfection Plus Ltd
Company Address	6 Westwood Court, Brunel Road, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

#### PRODUCT CODE- REF

PP013/0200	EasiJect Sterile Needles 27Gx21mm Pk 100 Imperial
PP013/0203	EasiJect Sterile Needles 27Gx35mm Pk 100 Imperial
PP013/0206	EasiJect Sterile Needles 30Gx12mm Pk 100 Imperial
PP013/0208	EasiJect Sterile Needles 30Gx21mm Pk 100 Imperial
PP013/0209	EasiJect Sterile Needles 30Gx25mm Pk 100 Imperial
PP013/0300	EasiJect Sterile Needles 27Gx21mm Pk 100 Metric
PP013/0301	EasiJect Sterile Needles 27Gx25mm Pk 100 Metric
PP013/0303	EasiJect Sterile Needles 27Gx35mm Pk 100 Metric
PP013/0306	EasiJect Sterile Needles 30Gx12mm Pk 100 Metric
PP013/0307	EasiJect Sterile Needles 30Gx16mm Pk 100 Metric
PP013/0308	EasiJect Sterile Needles 30Gx21mm Pk 100 Metric
PP013/0309	EasiJect Sterile Needles 30Gx25mm Pk 100 Metric
PP013/0310	EasiJect Sterile Needles 30Gx30mm Pk 100 Metric

#### PATIENT TARGET GROUPS

Paediatric to elderly patients who require injection of a local anaesthetic of a local anaesthetic for dental treatment.

#### INTENDED USERS

Dentists.


#### CONTRA INDICATIONS

Dental needle itself has no contra-indications.

#### PERFORMANCE CHARACTERISTIC

Injection of dental local anaesthetics.

#### METHOD FOR USE

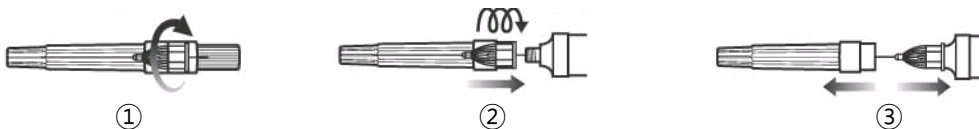

1. Break the hermetic seal (Twist cap to open – Do not bend or snap open).
2. Connect the needle on the dental cartridge syringe.
3. Remove the protective cap to use.

#### WARNING

- Do not use if seal is broken
- Do not reuse to avoid patient cross-infection
- Do not bend the needle; if the needle is bent, the needle can break in tissue and cause injury.
- After use - Do not touch the needle tip and dispose of it using the protective cap to prevent infection and being pricked.




















**STORAGE CONDITION**

Store at room temperature.

**CAUTION**

Note: If any serious incident that has occurred in relation to the medical device, please report it to the manufacturer and/or authorized representative and competent authority.

**EXPLANATION OF SYMBOLS USED**


	Catalogue number		Non Sterile
	Batch code		Avoid direct sunlight
	Use by date		Temperature limit
	Manufacturer		Caution
	Distributor		Medical Device
	Importer		Ec Rep
	Consult Instructions for use		UDI
	EO Sterilisation		Do not use if packaging is damaged
	Do not re-use		



C-K Dental Ind.  
88-50, Sudo-ro, Bucheon-si  
Gyeonggi-do  
Republic of Korea  
[ck-dental@hanmail.net](mailto:ck-dental@hanmail.net)  
Tel: +82-32-679-1204  
Fax: +82-32-679-1207



Perfection Plus Ltd  
6 Westwood Court  
Brunel Road, Totton  
Hampshire  
SO40 3WX, UK  
[www.perfectionplus.com](http://www.perfectionplus.com)

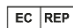

OBELIS S.A.  
Bd. Général Wahis, 53,  
1030 Brussels,  
Belgium  
Tel: +32-2-732-59-54  
Fax: +32-2-732-60-03  
Email: [mail@obelis.net](mailto:mail@obelis.net)



Perfection Plus EU Ltd  
The Black Church  
St Mary's Place.  
Dublin, Ireland,  
D07 P4AX  
[Regulatory@perfectionplus.com](mailto:Regulatory@perfectionplus.com)




1639