

ProphyPLUS Single Dose Instructions for Use

IMPORTANT

Read this 'Instructions for Use' document before handling. This product is for use by a dental professional only.
SPECIALLY FORMULATED FOR USE IN DENTISTRY

IDENTIFICATION

Product Name	ProphyPLUS Single Dose Prophy Paste pk 200 cups
Company Name	Perfection Plus Ltd
Company Address	6 Westwood Court, Brunel Road, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

PRODUCT CODE - REF

PP010/0100 Medium Mint
 PP010/0101 Coarse Mint
 PP010/0102 Medium Cherry
 PP010/0103 Coarse Cherry
 PP010/0104 Funky Flavours with 200 stickers

INDICATIONS FOR USE

A blend of polishing and cleaning agents designed for professional use during dental/orthodontic hygiene procedures. Each unit dose cup contains enough paste for a single prophylaxis treatment.

INSTRUCTIONS FOR USE

Using a prophy cup fill by pressing the face of the cup into the paste whilst running at a low speed. Maintain a quick and light touch. Sufficient paste should be maintained on the cup to avoid overheating of the tooth surface. Evacuation of saliva is recommended to avoid dilution and swallowing of the paste.

WARNING!

Paste should be expectorated and not swallowed.
 Product contains 1.23% sodium Fluoride, unless noted, nausea can result from excessive ingestion of fluoride.
 Keep out of reach of children.

Caution



Should a serious incident in relation to the product occur, it should be reported to the manufacturer and the competent authority.

STORAGE

Store in a cool dry place below 25°C. Keep out of direct sunlight.



Batch code/Expiry Date

See pot for Lot number and expiration date.

Distributed by: Perfection Plus Ltd, 6 Westwood Court, Brunel Road, Totton, Hampshire, SO40 3WX

 Dental Technologies
 6901 North Hamlin
 Avenue
 Lincolnwood
 Illinois 60712, USA
www.dentaltech.com

 Perfection Plus Ltd
 6 Westwood Court
 Brunel Road, Totton
 Hampshire
 SO40 3WX, UK
www.perfectionplus.com
Regulatory@perfectionplus.com

 Emargo Europe
 Westervoortsedijk 60
 6827 AT, Arnhem
 The Netherlands

 Perfection Plus EU Ltd
 The Black Church
 St Mary's Place.
 Dublin, Ireland,
 D07 P4AX
Regulatory@perfectionplus.com

