


## PoliSTRIP Plus

### Instructions for Use

#### **IMPORTANT**


Read this 'Instructions for Use' document before handling. This product is for use by a dental professional only. PoliSTRIP are used by the Dental professional for the finishing and polishing of composite restorations.

Product Name	PoliSTRIP Plus
Company Name	 Perfection Plus Ltd
Company Address	6 Westwood Court, Brunel Road, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

#### **PRODUCT CODE – REF**

PP010/1860 4mmx150mm Ultra Fine/Fine pk 50  
 PP010/1861 4mmx150mm Medium/Coarse pk 50

#### **STERILISATION**

The strips do not require sterilisation. The strips should be kept in their original packaging at room temperature and protected against dust and moisture until used. 

#### **STORAGE**

The strips should be kept in hygienic stands, dishes or other suitable containers and they must be protected against dust, moisture, and contamination during storage. If the strips are not used right away, it is advisable to keep them in their original packaging.

#### **PROPER USE**

- Products that are deformed should not be used and must be disposed of.
- Use the strips in the correct order, incorrect procedures can result in a reduction in the polishing quality.
- Wear a face mask to prevent biological transfer.
- Incorrect use produces poor results and increases the risk. These products must only be used by qualified staff.

#### **PRESSURE**

- Excessive pressure must be always avoided.
- Excessive pressure may cause heat build-up which may damage the strip.

Excessive pressure may cause overheating, which injures the pulp. In extreme cases, the instrument may even fracture

#### **INSTRUCTIONS**

1. Insert the central section (the area without abrasive) of the finishing strip between contact points.
2. Position the selected portion of the strip over the composite surface to be finished.
3. Firmly grasp both ends of the strip and draw the abrasive over the composite in a vigorous, back and forth motion
4. Repeat procedure using another grade of the strip.
5. Discard strip.
6. Repeat steps 1-5 with Polishing strips

#### **DISPOSE OF WORN INSTRUMENTS**

- Fractured and incorrectly shaped strips can cause injury.

#### **SAFETY PRECAUTIONS**

These dental instruments were developed and manufactured for their specific dental surgical application. Incorrect use may harm tissue, cause premature wear, destroy the instruments, and endanger the operator, patient or third parties.

 Stoddard  
 Manufacturing  
 Company Ltd  
 Black Horse Road  
 Letchworth  
 Herts, SG6 1HB, UK

 Perfection Plus Ltd  
 6 Westwood Court  
 Brunel Road, Totton  
 Hampshire  
 SO40 3WX, UK  
[www.perfectionplus.com](http://www.perfectionplus.com)

 Stoddard ApS  
 Moellegade 32  
 8000 Aarhaus C  
 Denmark

 Perfection Plus EU Ltd  
 The Black Church  
 St Mary's Place.  
 Dublin, Ireland,  
 D07 P4AX  
[Regulatory@perfectionplus.com](mailto:Regulatory@perfectionplus.com)

