

Protect+ 3-in-1 Tips Instructions for Use

IMPORTANT

Read this 'Instructions for Use' document before handling. This product is for use by a dental professional only. Protect+ 3-in-1 Tips are intended to be used to prevent cross contamination when using an air-water syringe.

Product Name	Protect+ 3-in-1 Tips
Company Name	Perfection Plus Ltd
Company Address	6 Westwood Court, Brunel Road, Totton, Hampshire, SO40 3WX, UK
Company Phone No.	+44 (0) 2380 866 677

PRODUCT CODE –

REF

PP002/5800 Protect+ 3-in-1 Tips White pk 250
 PP002/5801 Protect+ 3-in-1 Tips Multi coloured pk 250

STERILISATION

The tips do not require sterilisation before use and should be disposed of after use.



STORAGE

Store in a well-ventilated place, below 40C and less than 80° humidity. Keep away from acid and avoid direct sunlight during storage and transportation.

PROPER USE

- Wear a face mask to prevent biological transfer.
- Dispose of after use.

INSTRUCTIONS

1. Check the tip prior to use for obvious signs of contamination
2. Insert the long end of the tip into the Air/Water Syringe
3. Depending on the syringe being used, a converter may be required.
4. Once inserted, gently pull the tip to check it has been firmly installed.
5. To ensure the tips will spout water, gas or mist press the water and gas buttons either separately or simultaneously.
6. Dispose after use.



Wellmed Dental Medical
Supply Co., Ltd
F116, 4/F, Hehe Mansion
No. 68 Xin'an 2nd Road
Boan District
Shenzhen, China
www.wellmed.cn

Distributed by:
Perfection Plus Ltd
6 Westwood Court
Brunel Road, Totton
Hampshire
SO40 3WX, UK
www.perfectionplus.com



SUNGO Europe B.V
Olympisch Stadion
24
1076DE
Amsterdam
Netherlands



Perfection Plus EU Ltd
The Black Church
St Mary's Place.
Dublin, Ireland,
D07 P4AX
Regulatory@perfectionplus.com

