

PerfectaDAM


Light-Cured Liquid Dam




- Light reflecting resin barrier for isolating tissue adjacent to teeth
- Ideal for tooth whitening procedures and creating resin seal around dental dam clamps
- 4 x 1.2ml Syringes and 8 Tips

REF 004/0205


Technical Guide

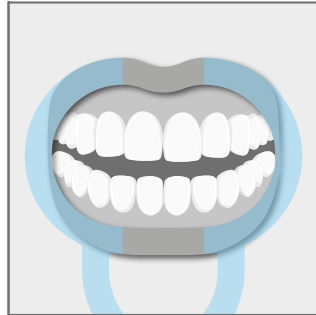


- 1 Place the retractor in the mouth

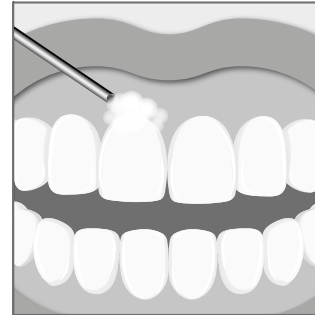


- 2 Wash thoroughly and dry the teeth

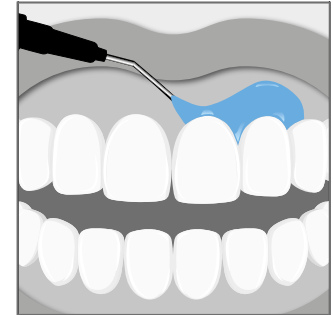


- 3 Apply PerfectaDAM in 4-6mm by incrementing 1.5-2mm in thickness

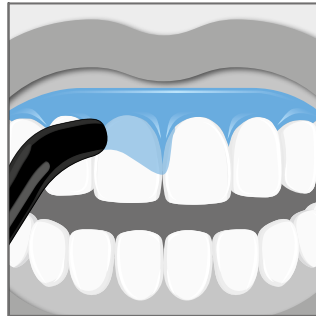

- 4 Light cure for 20 seconds

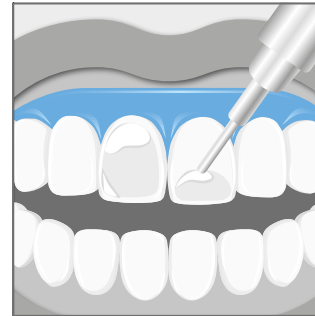

- 5 Apply whitening material and follow the manufacturer's instructions

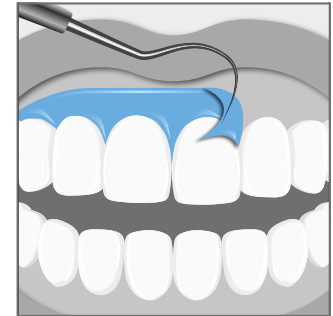

- 6 Remove the material easily in one piece after whitening procedure is complete



www.perfectionplus.com

PerfectaDAM

Light-Cured Liquid Dam




- Light reflecting resin barrier for isolating tissue adjacent to teeth
- Ideal for tooth whitening procedures and creating resin seal around dental dam clamps
- 4 x 1.2ml Syringes and 8 Tips


REF 004/0205

Technical Guide


Use with Dental Dam

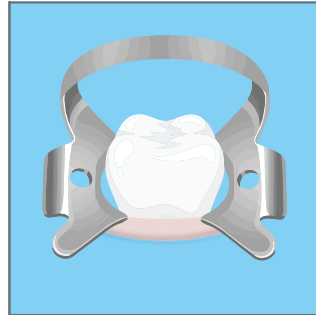


- 1** Isolate the working area with dental dam placement

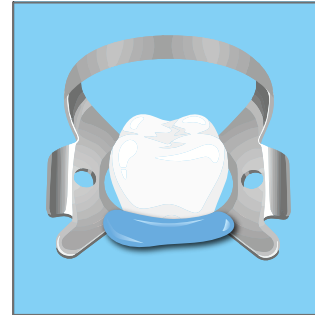

- 2** Apply a layer of PerfectaDAM in 1.5-2mm increments in the gap between dental dam and tooth

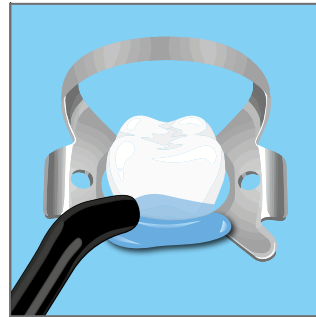

- 3** Light cure for 20 seconds before proceeding to treatment

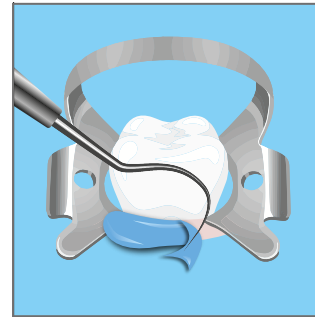

- 4** Remove the material easily in one piece after treatment is complete

Non Latex Dam

Light Blue Synthetic Polyisoprene



- Non-sterile, single use
- High Tensile strength
- 6 x 6 inches (152mm x 152mm)
- Medium 0.18mm - 0.23mm
- Heavy 0.23mm - 0.28mm

20 Pieces

REF 004/0046 Medium

REF 004/0047 Heavy

80 Pieces

REF 004/0050 Medium

REF 004/0051 Heavy



www.perfectionplus.com

Latex Sheets and full Dam Kits also available